Charger power pack user guide

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Thank you

Thank you for choosing the charger power pack. The intended use of the charger power pack is to offer additional charge to compatible hearing aids, without plugging in the charging case into an outlet.

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Hearing care professional: _____

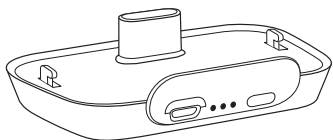
Telephone: _____

Serial number of charger power pack:

Warranty: _____


Date of purchase: _____

Quick reference

- ⓘ Before using your charger power pack for the first time, it is recommended to charge it for 3 hours.

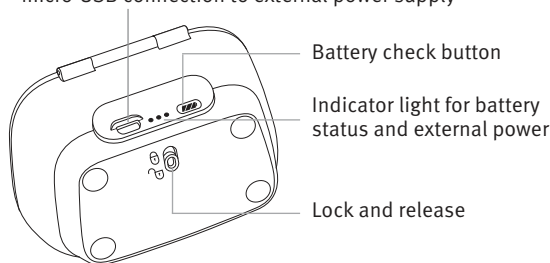micro-USB connection to external power supply

Indicator light

One green dot	•	0% – 29% charged
Two green dots	••	30% – 59% charged
Three green dots	•••	60% – 100% charged

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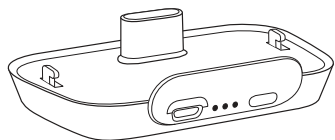
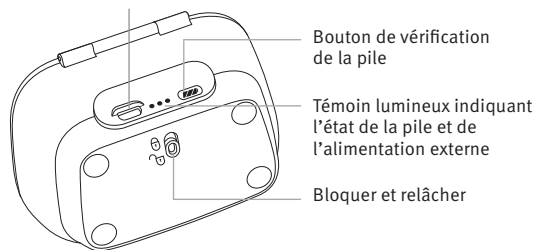
Table of contents

Using the charger power pack.....	2
1) Attach the charger power pack to the charging case and lock it.	2
2) Connect the power supply	3
3) Watch the indicator light.....	4
Troubleshooting guide	5
Warnings	6
Information and explanation of symbols.....	9
Compliance information.....	11
Your feedback	13
Additional notes.....	14

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
Using the charger power pack


- ① The charger power pack is only compatible with the charging case.

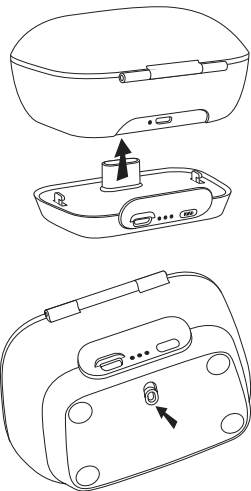- 1) Attach the charger power pack to the charging case and lock it.


- ① Before using the charger power pack for the first time, it is recommended to charge it for 3 hours.

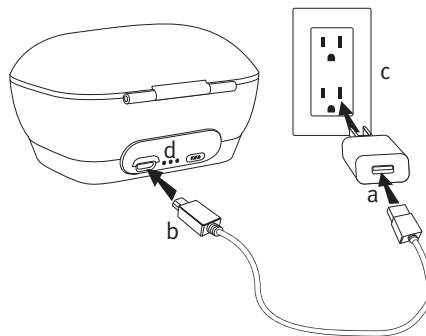- 2) Connect the power supply

- a) Plug the larger end of the charging cable into the power supply.
 - b) Plug the smaller end into the micro-USB port on the charger.
 - c) Plug the power supply into a power outlet.
 - d) The indicator light is green when the charger power pack is connected to the power supply.
- ① The charger power pack and hearing aids will be charged at the same time.

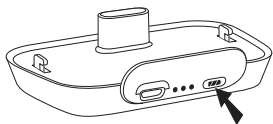The fully charged charger power pack offers 7 full charging cycles for a pair of hearing aids. To check the remaining battery capacity of the charger power pack, press the battery check push button (indicator lights will be off otherwise, to preserve battery).

3) Watch the indicator light

One green dot	•	0% – 29% charged
Two green dots	••	30% – 59% charged
Three green dots	•••	60% – 100% charged

- ① The charger power pack contains a built-in and non-removable rechargeable lithium-polymer battery.
- ① Please ensure to charge the charger power pack within operating temperature range: +5° to +40° Celsius (41° to 104° Fahrenheit)

Troubleshooting guide

Cause	Possible remedy
Charging does not start when using the charger power pack	
Charger power pack discharged	Charge the charger power pack, see “Using the charger power pack”

For any problems not listed, contact your hearing care professional.

Warnings

- ⚠ The charger power pack contains lithium-ion batteries that can be taken onto airplanes as carry-on luggage. The charger power pack cannot be placed in the checked-in baggage and can only be placed in the carry-on luggage.
- ⚠ Changes or modifications to the power pack that are not explicitly approved by the manufacturer are not permitted.
- ⚠ Do not use the charger power pack in explosive areas (mines or industrial areas with danger of explosions, oxygen rich environments or areas where flammable anesthetics are handled) or where electronic equipment is prohibited.
- ⚠ Do not cover the whole charger while charging, e.g. with a cloth etc.

- ⚠ Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- ⚠ Only use the charger and power supply provided by the manufacturer for charging. Otherwise, the device(s) may be damaged.
- ⚠ Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- ⚠ The USB port of the charging power pack is to be used for the described purpose only.

Precautions

- ① The hearing aids have to be dry before charging. Otherwise the charging reliability cannot be guaranteed.
- ① The charger power pack contains lithium-ion batteries with a Watt hour rating <20 Wh which have been tested according to UN 38.3 of the “UN Manual of Tests and Criteria” and should be shipped in accordance with all the rules and regulations on the safe shipment of lithium-ion batteries.

Labeling

The serial number and year of manufacture are located on the back of the charger power pack.

Information and explanation of symbols

With the CE symbol, Sonova AG confirms that this product – including accessories – meets the requirements of the Medical Devices Directive 93/42/EEC as well as the Radio Equipment Directive 2014/53/EU on radio and telecommunications transmitters.

This symbol indicates that it is important for the user to read and take into account the relevant information in this user guide.

This symbol indicates that it is important for the user to pay attention to the relevant warning notices in this user guide.

Important information for handling and effective use of the product.

Copyright symbol

This symbol shall be accompanied by the name and the address of the authorised representative in the European Community.

This symbol shall be accompanied by the name and the address of the manufacturer (who are placing this device on the market).

Operating conditions:

This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in these user guides.

Transport conditions:

Temperature: 5°C (41°F) – +40°C (+104°F).

Humidity: 0% – 75%

Pressure: 20kPA – 150kPA

The symbol with the crossed-out garbage bin is to make you aware that this device may not be thrown away as normal household waste. Please dispose of old or unused devices, at waste disposal sites intended for electronic waste, or give your device to your hearing care professional for disposal. Proper disposal protects the environment and health.

Compliance information

Declaration of conformity

Hereby, Sonova AG declares that this product meets the requirements of the Medical Device Directive 93/42/EEC and complies with Radio Equipment Directive 2014/53/EU. The full text of the Declarations of Conformity can be obtained from the manufacturer.

Australia/New Zealand:

Indicates a device's compliance with applicable Radio Spectrum Management's (RSM) and Australian Communications and Media Authority (ACMA) regulatory arrangements for the legal sale in New Zealand and Australia. The compliance label R-NZ is for radio products supplied in the New Zealand market under conformity level A1.

Notice 1

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- 1) this device may not cause harmful interference, and
- 2) this device must accept any interference received, including interference that may cause undesirable operation.

Notice 2

Changes or modifications made to this device not expressly approved by Sonova AG may void the FCC authorization to operate this device.

Notice 3

This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and ICES-003 of Industry Canada. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the equipment and receiver
- Connect the device into an outlet on a circuit different from that to which the receiver is connected
- Consult the dealer or an experienced radio/TV technician for help

Your feedback

Record your specific needs or concerns and bring to your first office visit after getting your hearing aids.

This will help your hearing care professional to address your needs.

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